

SINGLE FOLD BRIEFCASE MOBILITY RAMP FOR WHEELCHAIRS SCOOTERS

MR3, MR4, MR5, MR6

321300, 321400, 321500, 321600




Operator's Manual



Read the Operator's Manual entirely. When you see this symbol, the subsequent instructions and warnings are serious follow without exception. Your life and the lives of others depend on it!

TITAN
RAMPS

PARTS DIAGRAM / EXPLODED VIEW

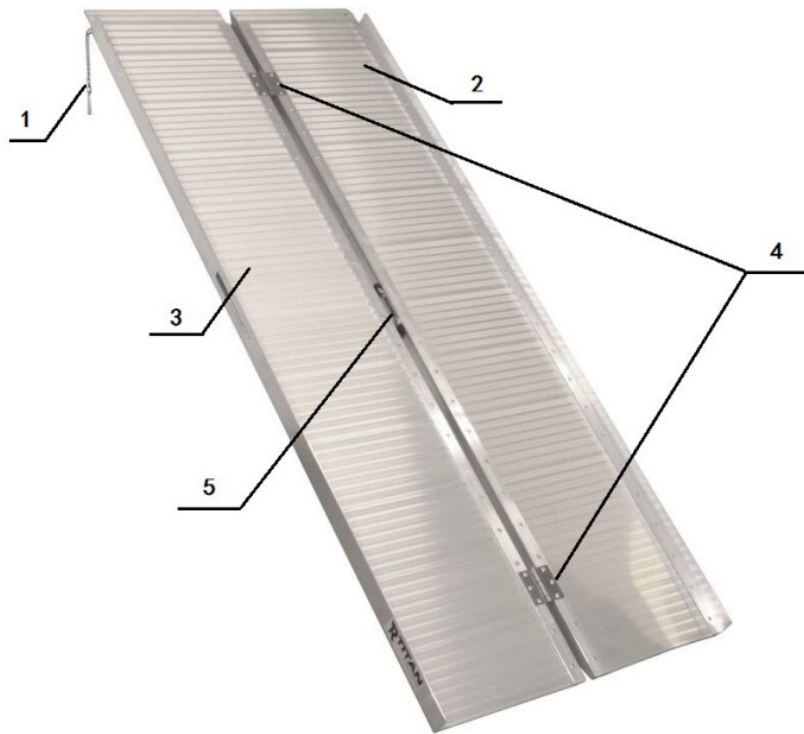

KEY	BOX	DESCRIPTION	QTY
(1)	1	CHAIN WITH FIXING PIN	2
(2)	1	SECTION A	1
(3)	1	SECTION B	1
(4)	1	HINGE	2
(5)	1	HANDLE	1

ASSEMBLY INSTRUCTIONS

NO ASSEMBLY IS NEEDED.

LOADING WHEELCHAIR:

- A. Position the wheelchair in the direction of the ramp with front wheel forward. Roll or ride the wheelchair slowly and steadily up the ramp, ensuring that the wheels follow the center of the ramp.
- B. Take extra care in wet conditions to ensure the wheels do not slide on the ramp.

UNLOADING WHEELCHAIR:

- A. Position the ramp as detailed above directly behind the rear wheel of the wheelchair. Allow the wheelchair to roll backwards down the ramp, ensuring the wheels follow the center of the ramp.
- B. Always have a qualified assistant present when using the ramp.

ADDITIONAL INSTRUCTIONS

MAXIMUM *OCCUPIED/**UNOCCUPIED HEIGHT LIMIT TO GROUND LEVEL:

2 FT Ramp: Occupied loading: 4-in maximum, unoccupied loading height: 6-in.

3 FT Ramp: Occupied loading: 6-in maximum, unoccupied loading height: 9-in.

4 FT Ramp: Occupied loading: 8-in maximum, unoccupied loading height: 12-in.

5 FT Ramp: Occupied loading: 10-in maximum, unoccupied loading height: 15-in.

6 FT Ramp: Occupied loading: 12-in maximum, unoccupied loading height: 18-in.

7 FT Ramp: Occupied loading: 14-in maximum, unoccupied loading height: 21-in.

8 FT Ramp: Occupied loading: 16-in maximum, unoccupied loading height: 24-in.

9 FT Ramp: Occupied loading: 18-in maximum, unoccupied loading height: 27-in.

10 FT Ramp: Occupied loading: 20-in maximum, unoccupied loading height: 30-in.

*Occupied is a person sitting in the wheelchair/scooter while loading (2:12 slope or 9.6-degree angle)

**Unoccupied is loading with nobody sitting in the wheelchair/scooter (3:12 slope or 14.5-degree angle)

MAXIMUM WEIGHT CAPACITY IS 600 LB

(WEIGHT RATING IS BASED ON EVENLY DISTRIBUTED LOAD ON 2 AXLES)

ACKNOWLEDGEMENT OF RISK AND RELEASE OF LIABILITY

The use of any equipment, including this one, involves the potential risk of injury. Apart from any warranty claim that might be presented for a claimed defect in material or workmanship of the product, you accept and assume full responsibility for any and all injuries, damages (both economic and non-economic), and losses of any type, which may occur, and you fully and forever release and discharge Titan, its insurers, employees, officers, directors, associates, and agents from any and all claims, demands, damages, rights of action, or causes of action, present or future, whether the same be known or unknown, anticipated, or unanticipated, resulting from or arising out of the use of said equipment.

This equipment must be used with care by capable and competent individuals under supervision, if necessary.

TITAN LIMITED WARRANTY: TERMS, EXCLUSIONS AND LIMITATIONS OF REMEDIES

This product comes with a one (1) year limited warranty that can be found at www.titanramps.com/warranty.html. Please review the same for all details regarding the Titan Limited Warranty.

THE TITAN LIMITED WARRANTY FOUND AT WWW.TITANRAMPS.COM/WARRANTY.HTML IS EXCLUSIVE AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE, EACH OF WHICH IS HEREBY DISCLAIMED.



NEED HELP? CONTACT US FIRST.

1-800-605-7555

info@titanramps.com

www.titanramps.com

© 2021 Titan Brands